## INTRODUCTION

Thank you for purchasing the ReliOn® H-003DREL Extra Large Cuff. This cuff is designed to meet the needs of individuals with an arm circumference size of 13” to 17”. The ReliOn® Extra Large Cuff is intended for use with ReliOn® and Omron® digital blood pressure monitors sold with the D-ring style cuff.

## HOW TO USE

Disconnect the existing arm cuff on the monitor. To disconnect gently pull on the air plug. **DO NOT** pull on the air tube.


Connect the air plug from the large cuff into the air jack of the blood pressure monitor.

Refer to the Operating Instructions in the manual included with your blood pressure monitor for information on **Applying the Arm Cuff and Taking a Reading**.

## IMPORTANT SAFETY NOTES

- **AIR RELEASE VALVE**

DO NOT BLOCK THE END OF THE SHORT TUBE THAT CONTAINS THE SUPPLEMENTAL AIR RELEASE DEVICE. THE DEVICE IS NECESSARY TO MAINTAIN A PROPER DEFLATION RATE. BLOCKING THIS TUBE MAY RESULT IN AN ERRONEOUS READING.

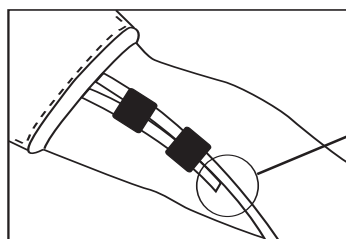DO NOT BLOCK THIS PART

- **MONITOR WITH AUTOMATIC INFLATION**

IF THE UNIT TAKES MORE THAN 25 SECONDS TO INFLATE THE CUFF, THE BATTERIES MAY NEED TO BE REPLACED. WEAK BATTERIES WILL EXTEND THE INFLATION TIME AND MAY CAUSE SOME ARM DISCOMFORT. WE RECOMMEND THE OPTIONAL AC ADAPTER BE USED WITH THE LARGE CUFF.

## SPECIFICATIONS

Arm Circumference Range: Approximately 13” to 17” (32-42 cm)

Cuff Dimensions: 24 1/2 ” x 6 7/8”

## WARRANTY INFORMATION

Your ReliOn® H-003DREL Extra Large Cuff is warranted to be free from defects in materials and workmanship appearing within one year from the date of purchase, when used in accordance with the instructions provided with the unit. The above warranties extend only to the original retail purchaser. We will, at our option, repair or replace without charge any cuff covered by the above warranties. Repair or replacement is our only responsibility and your only remedy under the above warranties.

To obtain warranty service contact Omron Healthcare for the address of the repair location. Shipping instructions and repair information is available by calling **1-800-634-4350** or visiting our website at [www.omronhealthcare.com](http://www.omronhealthcare.com).

Enclose the Proof of Purchase. Include \$5.00 for the return shipping and handling. Include a letter, with your name, address, phone number, and description of the specific problem. Pack the product carefully to prevent damage in transit. Because of possible loss in transit, we recommend insuring the product with return receipt requested.

**ALL IMPLIED WARRANTIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR PARTICULAR PURPOSE, ARE LIMITED TO THE DURATION OF THE APPLICABLE WRITTEN WARRANTY ABOVE.**

Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

**OMRON SHALL NOT BE LIABLE FOR LOSS OF USE OR ANY OTHER INCIDENTAL, CONSEQUENTIAL OR INDIRECT COSTS, EXPENSES OR DAMAGES.** Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above exclusions may not apply to you.

This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state.

### **FOR CUSTOMER SERVICE**

Visit our web site at:

[www.omronhealthcare.com](http://www.omronhealthcare.com)

Call toll free:

1-800-634-4350

Distributed by:  
**OMRON HEALTHCARE, INC.**  
1200 Lakeside Drive  
Bannockburn, Illinois 60015  
[www.omronhealthcare.com](http://www.omronhealthcare.com)  
Made in China  
H-003DRELINST RevB

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