Pen Needles

Instructions for use

Tips for safe and comfortable injections inside!

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Compatible with all major diabetes pens, including:

* All brands are trademarks of their respective manufacturers.

** The Novo Function Check cannot be performed with this pen needle.

| PEN NAME* | PEN MANUFACTURER |
|---------------------------------|------------------------------|
| OptiClik® | Sanofi -Aventis |
| SoloStar® | Sanofi -Aventis |
| NovoPen® 3, 4, & Junior** | Novo Nordisk A/S |
| FlexPen®** | Novo Nordisk A/S |
| Autopen® | Owen Mumford, Inc. |
| Humulin® Pen | Eli Lilly and Company |
| Humalog® Pen | Eli Lilly and Company |
| HumaPen® Ergo, Luxura, & Memoir | Eli Lilly and Company |
| Byetta Pen™ | Amylin/Eli Lilly and Company |
| SymLinPen™ | Amylin Pharmaceuticals, Inc. |

Consult your physician and follow your pen's instructions for use before using these pen needles. Please contact us if you have any questions.

Remove pen needle seal.

Click pen needle onto pen, by pressing straight down using the outer protective cap*.

Do not click on at an angle.

* Note: The pen needle can also be twisted on; a secure fit is ensured even if there is no noticeable stop

Remove the outer protective cap by pulling it straight off; save it.

Remove the inner protective cap.

Penetrate skin quickly and inject slowly according to your physician's and pen's instructions for use.

Carefully replace the outer protective cap (if self-injecting only).

Unscrew the pen needle and safely dispose of it according to your physician's or pharmacist's advice.

Warnings:

AVOID INFECTIONS:

- Do not use the pen needle if the seal is damaged or has been opened by someone other than you.
- Do not use the pen needle after the expiration date on the packaging has expired.
- Before each injection, clean the injection site thoroughly according to your physician's instruction.
- Use each pen needle only once.
- Do not re-use, clean and/or re-sterilize pen needles.
- Carefully replace the outer protective cap on the

used pen needle and dispose of it immediately after self-injection. Follow your physician's or pharmacist's advice for proper disposal in your area.

- If these instructions are not followed, there is a risk of infection.
- Seek medical attention if you have any health concern.

NEVER HANDLE SOMEONE ELSE'S PEN NEEDLE

- To avoid accidental needlesticks, which can transmit serious infections, never recap, remove, or destroy a pen needle that has been used by someone else.

Warnings:

GET THE CORRECT DOSAGE

- Consult your physician for the right needle length for you.
- Prime your pen according to your pen's instructions for use before each injection to ensure that the needle allows free flow of medication.
- Inject slowly according to your physician's advice and pen's instructions for use.
- Check after each injection whether medication drips from the pen needle, pen, or injection site. If the medication drips, follow your physician's

instructions regarding the handling of potential under-dosage.

- Seek medical attention if you have any health concern.

AVOID BENDING OR BREAKING THE NEEDLE

- Do not change the direction of the pen and pen needle during insertion into or withdrawal from the skin.
- Seek immediate medical attention if a broken needle becomes stuck in the body or disappears under the skin.

TIPS FOR A SAFE AND COMFORTABLE INJECTION

- Change the injection site regularly in accordance with your physician's advice. If this is not done, tissue hardening may result.
- Inject medication at room temperature.
- Make sure no air bubbles remain in the pen before injection.
- Wait until topical alcohol (if used) has evaporated completely before injection.
- Keep muscles in the injection area relaxed when injecting.
- Penetrate the skin quickly.
- Do not change the direction of the needle during insertion into or withdrawal from the skin.
- Do not re-use pen needles.
- ! Please note the special instructions for employees in health systems.

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trademark used under license by
Can-Am Care LLC,
Can-Am Care, LLC.