## INSTRUCTION MANUAL

# Instant Ear Thermometer

Please read this detailed guidebook completely before operating this unit.

*English • Spanish*

## Limited Five-Year Warranty

Your Ear Thermometer is guaranteed to be free of manufacturing defects for a period of five years from the date of purchase under normal use. This limited warranty does not apply to use in any clinical or commercial applications. If the unit fails to operate during the warranty period, return it postage prepaid along with \$5.00 for return shipping and insurance to: Repair Department, 1931 Norman Drive South, Waukegan, IL 60085. When returning a product, please also include a letter with your name, address, phone number and brief description of the specific problem. Please carefully package the product to avoid any damage that may occur while in the transit. Packaging insurance with returned receipt is recommended. If the warrantor determines that the unit failed to operate due to a manufacturing defect, the unit will be repaired or replaced at the option of the warrantor. Repair or replacement of the unit is the sole remedy under this limited warranty. This warranty gives you specific legal rights, which vary from state to state. As a condition of this warranty, the enclosed warranty registration card must be completed and sent to us within 10 days of the purchase date.

This Limited Warranty constitutes the warrantor's only responsibility and obligation to repair and/or replace materials or components. We make no other express or implied warranties, arising by operation of law or otherwise, or any warranty of merchantability or fitness for a particular use or purpose whether or not the use or purpose has been disclosed to the warrantor in specifications, drawings or otherwise, and whether or not the warrantor's products are specifically designed and/or manufactured by the warrantor for the buyer's use or purposes, except for the limited warranty stated above. The warrantor will not be responsible for any indirect, incidental, special, consequential, or punitive damages or other loss, including, but not limited to, damage to or loss of other property or equipment and personal injuries, whether to purchaser or others. The warrantor shall in no event be liable to the purchaser for any amount in excess of the cost of repair and/or replacement of the unit.

TOLL-FREE CUSTOMER CARE HELPLINE  
**1-800-622-4714**

Manufactured for:  
**MABIS Healthcare**  
1931 Norman Drive South  
Waukegan, IL 60085 USA  
www.mabis.net  
E-mail: productinfo@mabis.net  
Phone: 847-680-6811  
Fax: 847-680-9646

## Index

Helpful Hints	3
Introduction	4
Product Features	5
Nite-Glo™ Display	5
General Care & Important Information	6
Cleaning & Storage	7
Taking a Reading	8
Changing Modes	9
Recalling Measurements in Memory	9
Replacing the Battery	9
Symbols	10
Troubleshooting	11
Product Specifications	12

## Helpful Hints

- For infants, it is best to position infant flat with the ear facing upwards.
- For children and adults, it is best to stand behind and slightly to the side.
- Always take measurements in the same ear since temperature readings may differ from ear to ear.
- Readings taken while a child is sleeping should not be compared to readings taken while awake.
- It is recommended to take three readings in the same ear with the highest interpreted as the reading for:
  - newborn infants in the first 100 days,
  - children under three years of age with compromised immune systems or with conditions where the absence of fever is critical,
  - a new user of an Instant Ear Thermometer until proper technique is established.

## Introduction


Thank you for purchasing an Instant Ear Thermometer. Please read the following instructions to achieve the most accurate temperatures and safe operation. With proper care and use, your thermometer will provide you with many years of reliable readings.

Your Instant Ear Thermometer is intended for monitoring human body temperature in the home and is appropriate for all ages. It is less threatening than a rectal thermometer, is easier and faster than standard oral measurements and can even be used while a child is sleeping.

In one second, 512 readings are taken. These readings are converted into a single temperature with the peak temperature displayed on the screen.

**NOTE:** Measurements taken with an Instant Ear Thermometer are comparable to oral readings.

## Product Features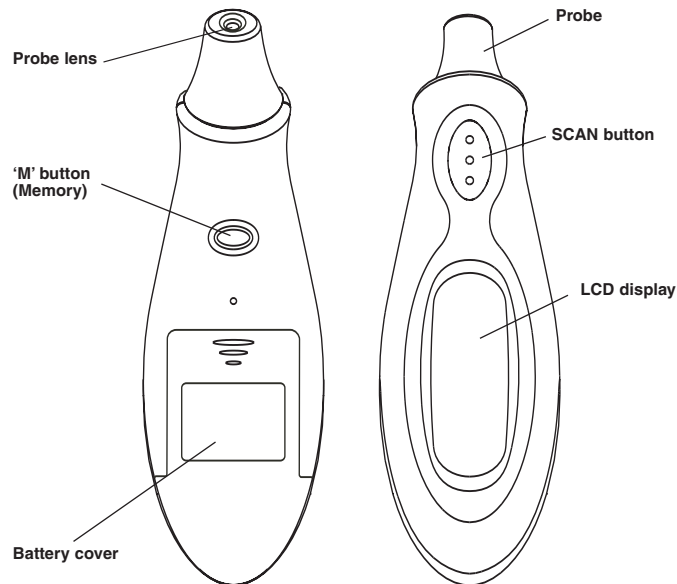

### Nite-Glo™ Display

Your Instant Ear Thermometer features an illuminating digital display. The display will illuminate for approximately 5 seconds every time the SCAN button is pressed to take a measurement. After each measurement is complete, the display will remain illuminated for approximately 5 seconds. The display will also illuminate for approximately 5 seconds every time the 'M' button is pressed to recall a reading in memory.

## General Care & Important Information

Your Instant Ear Thermometer measures the heat radiated from the eardrum and the surrounding tissue. This energy is transferred through the probe and converted into a single temperature value. Although readings taken from the eardrum (tympanic membrane) can provide the most accurate readings, improper technique can generate lower readings, resulting in a misdiagnosed fever.

1. The thermometer should only be used under the supervision of an adult.
2. Earwax in ear canal may cause an inaccurate reading. Consult your healthcare professional for proper removal and ear cleaning procedures.
3. Do not use excessive force to insert probe into ear.
4. Do not use if ear canal is wet or if infection is present.
5. Ear measurements are comparable to oral measurements. Remember temperatures can vary from 97.0°F to 99.0°F (36.1°C to 37.2°C) and still may be considered 'normal'. Time of day, level of activity, exercise and other activities can attribute to variations in temperature. However, unlike oral temperatures, factors such as smoking, talking and drinking do not affect ear measurements.
6. Do not walk, run or talk during temperature taking.
7. Clean the probe tip before and after each use.
8. Store the unit in the protective case when not in use.
9. Do not touch the probe with fingers.
10. Do not store the unit where it will be exposed to direct sunlight, dust or humidity. Avoid extreme temperatures.
11. Dropping or subjecting your thermometer to strong shocks should be avoided.
12. Do not attempt to disassemble the unit, except to replace the battery.
13. If measurement is not attainable, the unit must be replaced.

**NOTE:** Performance of the device may be degraded if: operated outside of stated temperature and humidity range; stored outside of stated temperature and humidity range; thermometer undergoes mechanical shock (drop); patient temperature is below ambient temperature.

## Cleaning and Storage

To ensure accurate measurements, clean the probe lens before and after each use. Fingerprints and earwax can cause inaccurate readings.

### To Clean:

- Use an alcohol swab or a soft cotton swab moistened with alcohol (70% Isopropyl) to the thermometer casing and probe lens, *Figs. 1-3*. Do not use paper towels. The fibers may scratch the lens.
- Make sure that liquid does not enter the interior of the thermometer.
- Never use abrasive cleaning agents, thinners or gasoline for cleaning.
- Do not immerse into water or other cleaning liquids.

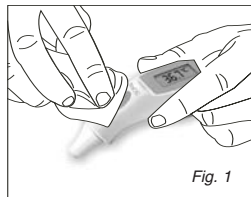
Fig. 1

### To Store:

- Remove the battery if the thermometer will not be used for an extended period of time.
- Store in a sun-free and dust-free, dry area at a temperature between 68°F – 122°F (20°C – 50°C).
- Protect the thermometer from severe impact or shock. Keep stored in the protective storage case.

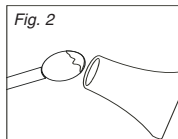
Fig. 2

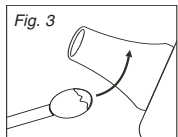
Fig. 3

## Taking a Reading

**NOTE:** Many pediatricians recommend the use of ear thermometers for children older than 6 months.

1. Press and release the SCAN button. After two seconds, the system will perform a self-test, *Fig. 1*, and show the last temperature taken along with a flashing "°F" to indicate the unit is ready for measurement, *Fig. 2*.
2. Gently insert the probe snugly into the ear canal until a slight resistance is felt. The thermometer will beep when the auto-position sensor detects proper placement. If the unit does NOT beep, gently reposition the thermometer.

**NOTE:** To disable the auto-position sensor, immediately press the SCAN button after inserting the thermometer into the ear canal.

3. After the unit beeps, press the SCAN button, *Fig. 3*, and promptly release the button. The device will beep once again after the measurement is complete.

**NOTE:** If you do NOT press the SCAN button once it is placed in the ear canal, the thermometer will automatically take a measurement in 9 seconds.

4. If the measurement is below 100.4°F (38°C), the thermometer will beep once and the measurement will be shown on the display with a happy face symbol, *Fig. 4*.
5. If the temperature is 100.4°F (38°C) or above, the thermometer will sound one long beep followed by two short beeps. The measurement will be shown on the display with a sad face symbol to indicate fever, *Fig. 5*.
6. Thermometer will automatically beep and shut off after approximately 1 minute of non-use.

**NOTE:** To ensure accuracy,

1. Please wait at least 10 seconds between successive readings.
2. Clean the probe lens before and after each use.

Fig. 1

Fig. 2

Fig. 3

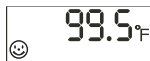
Fig. 4

Fig. 5

## Changing Modes

Temperature readings can be displayed in either Fahrenheit or Celsius. To switch between °F or °C, press and hold the SCAN button. Without releasing, also press the 'M' button and toggle between the modes, *Fig. 1 and 2*.

Fig. 1

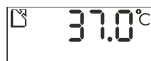
Fig. 2

## Recalling Measurements in Memory

The last 10 measurements taken are automatically stored into memory. To recall measurements, press the SCAN button to turn the unit on. After the unit beeps, press the 'M' button to view measurements in memory. Continue to press the 'M' button to view the next previously stored measurement, *Fig. 1*. The unit will beep each time a recorded measurement is viewed.

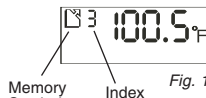
Fig. 1

Memory Symbol

Index


## Replacing the Battery

Your Instant Ear Thermometer requires one lithium battery, type CR2032. The battery needs to be replaced when the low battery symbol appears on the display.

To replace the battery:

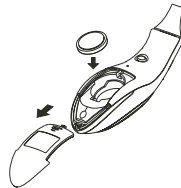- 1) Slide the battery cover off.
- 2) Using a non-metal pointed tool, carefully remove the battery and insert the new battery with the positive (+) side facing up.
- 3) Replace the battery cover.

**NOTE:** Please properly dispose the battery away from small children and heat.

## Symbols

Fahrenheit/Celsius	°F / °C
Memory symbol and measurement number	 8
Scanning in progress	
Low battery	
Happy face Indicates temperature is lower than 100.4°F (38.0°C)	
Sad face Indicates temperature is higher than 100.4°F (38.0°C)	

## Troubleshooting

Display Symbol	Condition/Cause	Corrective Action
	Measurement is below normal temperature range.	Retake temperature positioning the probe snugly into the ear canal.
	Measurement is above normal temperature range.	Retake temperature positioning the probe snugly into the ear canal.
	Operating temperature is outside of the permissible range (60.8°F – 104.0°F; 16.0°C – 40.0°C)	Take thermometer to a room where the temperature is within the permissible range. Wait 45 minutes to stabilize thermometer. Retake temperature positioning the probe snugly into the ear canal.
	Blank display	Check battery polarity or voltage. May need to replace battery.
	Low battery	Replace battery immediately.
	Low battery warning	Unit will work; however, replace battery when possible.

## Product Specifications

<b>Name and model number</b>	Instant Ear Thermometer, 144-107-000
<b>Display system</b>	Liquid Crystal Display with 0.1°F (0.1°C) resolution
<b>Accuracy</b>	±0.4°F, 96.8°F – 102.2°F (±0.2°C, 36.0°C – 39°C) ±0.5°F, <96.8°F or >102.2°F (±0.3°C, <36.0°C or >39°C)
<b>Measuring range</b>	93.2°F – 109.4°F (34.0°C – 43.0°C)
<b>Memory</b>	Automatically stores up to 10 readings.
<b>Tone</b>	One short beep sounds when the unit is turned ON. After the measurement is complete, one beep sounds for temperatures under 100.4°F (38.0°C). One long beep followed by two short beeps sound for temperatures over 100.4°F (38.0°C). One short beep sounds for temperatures <93.2°F (34°C) Lo or >109.4°F (43°C) Hi. Thermometer beeps once for each memory recall and when unit automatically turns off. Short rapid continually beeps indicate battery is low.
<b>Operating temperature</b>	60.8°F to 104.0°F (16.0°C to 40.0°C) at relative air humidity (not condensing) of up to 95%
<b>Storage temperature</b>	-4.0°F to 122.0°F (-20.0°C to 50.0°C)
<b>Automatic power-off</b>	Automatic shut off after approximately 1 minute of non-use.
<b>Battery</b>	1 each, CR2032 battery
<b>Dimensions</b>	4-1/2" (L) x 1-3/8" (W) x 1" (H) (11.5cm x 3.5cm x 2.5cm)
<b>Weight</b>	With battery, 1.6 ounces (45 grams)
<b>Accessories</b>	Storage case, Instruction Guidebook, Quick Start Guide, one CR2032 battery
<b>Standards</b>	Complies with US ASTM E-1965-98.

Specifications are subject to change without notice.

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©2006 ReliOn®